nospi
tali
beds
for
ICU
wards

IT - EN
ES - FR

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EN Welcome to the world of Malvestio, the Italian company that has been at the forefront of hospital innovation for more than 80 years with a team of 200 professionals and a presence in over 70 countries around the world. A company that is growing in size guided by a clear corporate vision: to offer healthcare increasingly advanced technologies, high production standards, organizational efficiency, high quality products and services.

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EN All Malvestio products, are created in accordance with the principles of the **Care Manifesto**, with which we have made our commitment to efficient healthcare focused on the quality of life of the patient. In particular, **"CARE FOR QUALITY"** is the duty we take on towards the efficiency of our work and our products, with which we contribute to the progress of healthcare and society.

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In intensive care, **technology is vital**

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EN Because VIVO is the best in our technology in facilities where therapeutic response is vital, it is called: intensive care. From the intuitive design to its range of functions, it is the choice of facilities that want to offer the highest level assistance.

VIVO is entirely electric, in 4 sections with adjustable height, with intuitive icons commands, touch screen and emergency controls that are always easy to reach. It is designed as a truly integrated bed-mattress system, capable of detecting and distributing pressure according to the inclination of the bed.

Thanks to the sensors, the **WEIGHING SYSTEM** can accurately weigh the patient, thus offering valid organisational support to operators.

In their activities, operators can count on the exclusive **PAS SYSTEM**, which reduces pressure for the patient and consequent slipping.

The **ROTATIONAL SYSTEM** controls are easily accessible so that the rotational system can be programmed at any time for patients with pulmonary complications.

The **DRIVESYSTEM**, is another essential aid for staff that has a motorized system that supports movement during transfers.

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optimal position for the patient
and less effort for the operators¹

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EN The innovative patented PAS system considerably reduces repositioning operations thanks to a backrest adjustment system that **works in synergy with the air mattress**.

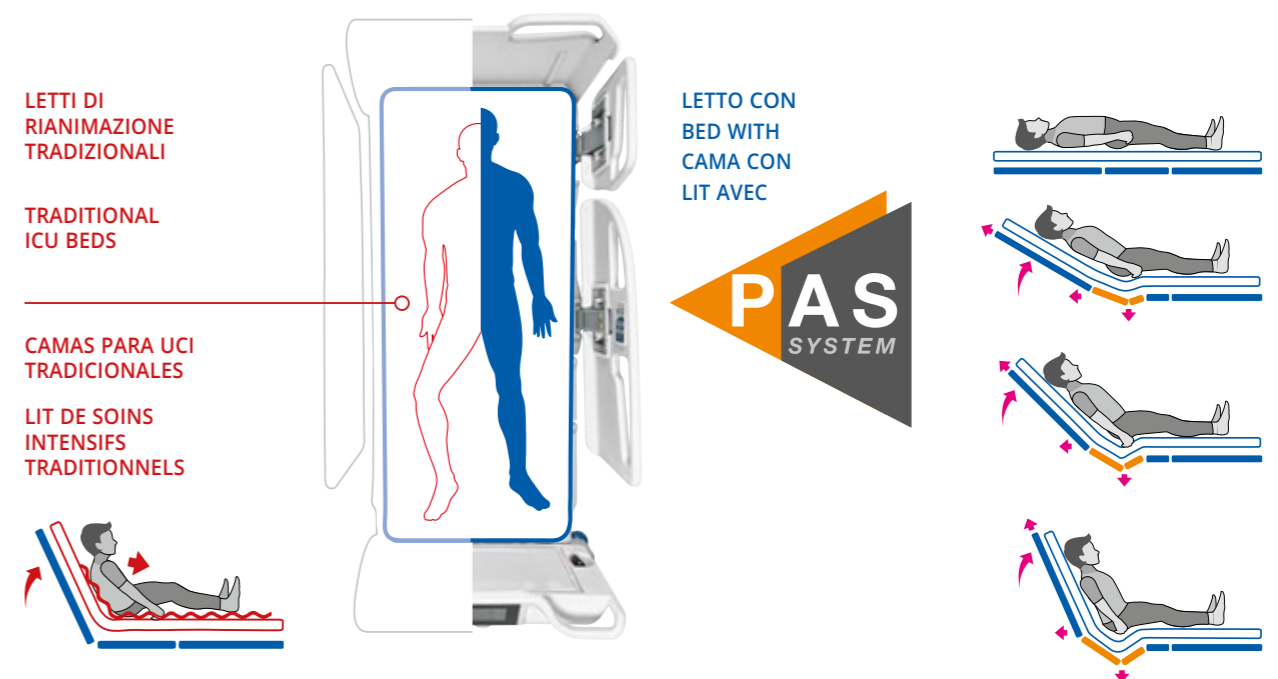
By preventing the patient from slipping towards the foot-end of the bed, also **shear stresses are prevented to superficial and deep skin tissues**, which are typically linked to the probability of **developing decubitus ulcers**. During adjustment, the backrest performs three simultaneous operations: it **rotates, translates and pulls upwards**, while the air mattress, which is divided into three sections, enhances those operations by moving together with the mattress platform, **preventing the compression of the pelvis area and self-centering the patient on the mattress**.

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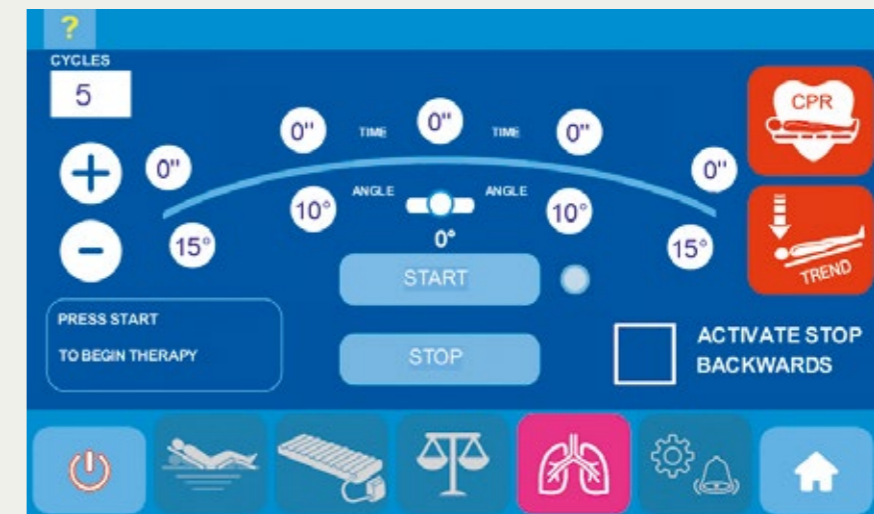
EN The **lateral tilt** allows to adjust the mattress platform at an angle of +/- 15° with intermediate steps of +/-5°, promoting breathing and oxygenation in the patient, contributing at the same time to **preventing respiratory complications**.

From the touch screen console it is possible to program a **personalized rotational system**, by setting up the number of cycles, intervals and angles.

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Tilting and weighing system
independent from the mattress

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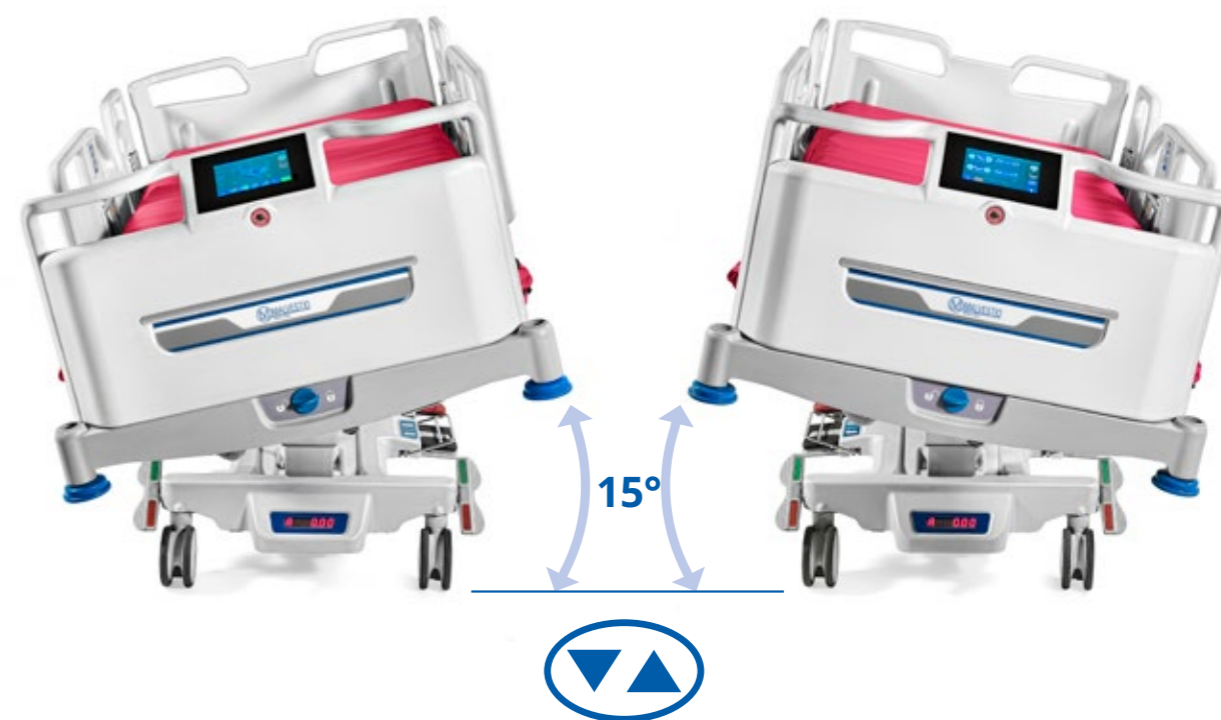
EN The inflating system of the **VI•MAT** mattress is incorporated into the footboard in order to facilitate Service operations.

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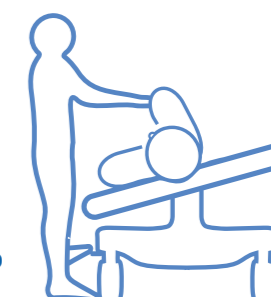
EN Moving the patient to bed, especially if he cannot cooperate, is always a delicate time for hospital staff. Tilting the lying surface makes the activity easier even for an individual operator. With Vivo, you can activate the **Lateral Tilt** from any side: from the touch console, from the push-button panel on the sides or from the pedals.

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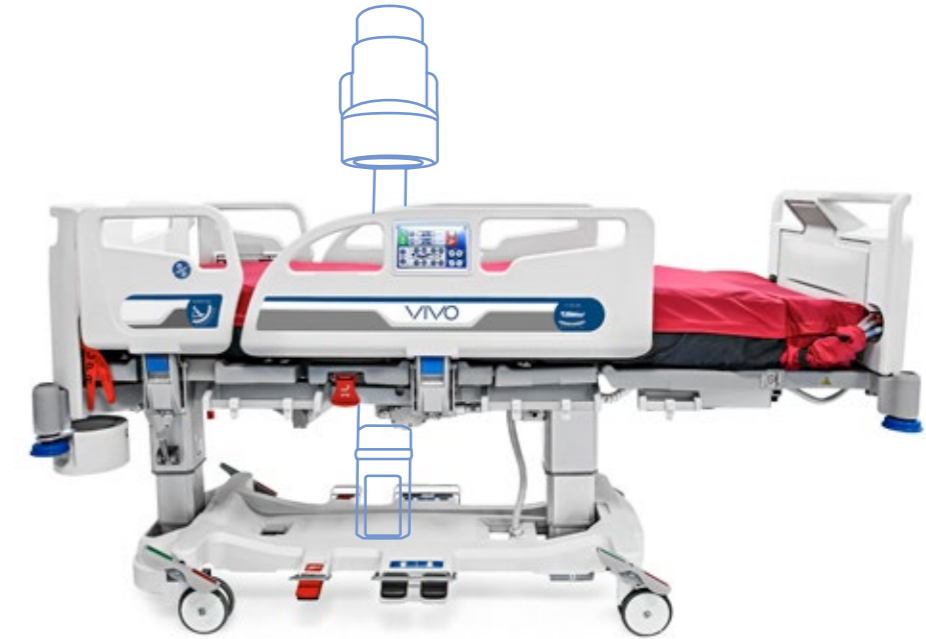
EN VIVO allows for a broad area of radiotransparency, thus allowing for the use of the most common equipment such as the image intensifier. The backrest can be equipped with an **x-ray cassette holder** for chest x-ray examinations, without moving the patient and at any angle of the backrest.

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EN In addition to the digital touch screen, the bed can be operated by means of **controls located on the outside of the safety sides**, featuring **LED lights for night vision**.

The LED light flashes in case of alert or nurse call.

The patient controls located on the inside of the safety sides can be inhibited by the nursing staff.

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Patient controls with nurse call through **SOUND AND VISUAL ALERT (WHICH CAN BE DISABLED)**.

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Dedicated height adjustment control available for **ASSISTED PATIENT EXIT**.

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EN Operator controls integrated in the safety sides with LED lights for **NIGHT VISION**.

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EN The exclusive **WEIGHING SYSTEM** was studied to guarantee a high level of precision. Thanks to the **4 sensors** located at the corners of the bed, the load is better distributed and the reading level more accurate. It is possible to **record patient weight data** and to view the corresponding graphics on the **touch screen**, with the measurements available both in kg and in Lb. While the weight of the patient is monitored it is possible to set up different types of **alert: exceeded threshold weight alert, 'bed exit' alert, 'restless patient' alert**.

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EN Thanks to the support, the ascent and descent brake assist of the **DRIVESYSTEM** moving system, I can easily face ramps with up to a 10% incline.

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Intensive and **semi-intensive care**
has just become more efficient

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EN ARIA combines Malvestio technologies into a flexible bed, suitable for both intensive and semi-intensive care.

Its functions respond to the various challenges of intensive care and promote safety and comfort for patients and staff.

The large area of radiotransparency allows for radiographic testing without moving patients from bed.

With the **SISTEMA INCENTRO** it is possible to prevent the patient's body from slipping. Furthermore,

through the **Lateral Tilt system**, it is easier to monitor the patient.

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
offsets the patient push towards the foot-end of the bed

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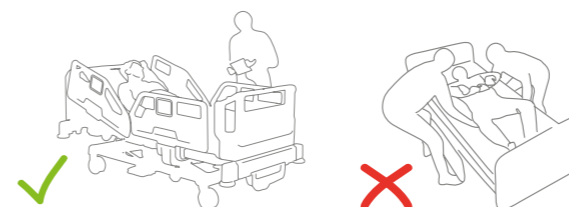**NO STRESS NO HASSLE, MORE TIME FOR
WHAT MATTERS MOST.**

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EN With the **SISTEMA INCENTRO**, the combined backrest-pelvis transfer cancels horizontal thrusts, prevents slipping and keeps the torso at an inclination of 30°.

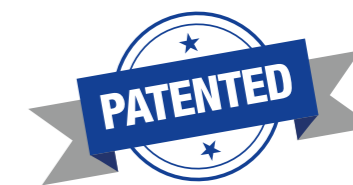
Intensive care patients need to be frequently repositioned by the nursing staff as their body tends to slip towards the foot-end of the bed, with subsequent skin friction. A non-slipping patient reduces fatigue and the risk of injury, and frees up staff to focus on activities of greatest value to patients.

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programmed rotation for patient health²

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EN The lateral tilting function can also be programmed from the touch screens to activate automatically, through the "rotational system" function. **The automatic inclination allows**, without engaging healthcare workers in strenuous activities, **for preventing complications deriving from a prolonged stay in the same position** such as pressure ulcers and intrapulmonary fluid formation.²

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EN Touch screen programming of **rotational system**.

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**LATERAL TILT: FOR EASIER PATIENT CARE AND LESS
FATIGUE FOR US PROFESSIONALS.**

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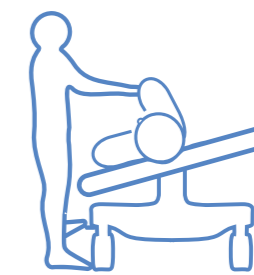
EN The lateral tilt of the mattress platform makes medical staff activities more ergonomic as the patient **can be placed on one side with the minimum effort**. This way, nursing staff can operate on the patient, preventing injuries and reducing fatigue at work. **The activation and extent of the lateral tilting can be controlled via the touch screen console, the side push button panel on the side rails and the pedal panel on both sides.**

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Controlling the patient's weight and movements in real time

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EN The exclusive weighing system was designed to guarantee a high level of precision, regardless of the position of the mattress platform. Thanks to the 4 sensors located at the corners of the bed, the load is better distributed and the reading level more accurate. The weighing system allow to set up different types of alerts.

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EN It is possible to record patient weight data and to view the corresponding graphics on the touch screen.

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EN With a large radiotransparent area, **ARIA** range beds are equipped for taking X-rays without having to remove the patient. The radiotransparent plastic laminate backrest features a cassette for X-ray plates that allows X-ray examinations to be carried out on a bedridden patient.

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EN **ARIA** allows for a broad area of radiotransparency, thus allowing for the use of the most common equipment such as the image intensifier.

The X-ray cassette holder can also be removed with the backrest in a horizontal position, thus avoiding disturbing the patient at the end of the examination.

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Safe maneuvers with just one operator

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EN Thanks to the **DRIVESYSTEM** ascent support and descent assisted braking system, ramps with inclines of up to 10% can be navigated easily and safely.

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EN COMPACT DIMENSIONS AND EASY MANOEUVRING

The practicality and safety of manoeuvring ARIA range beds are guaranteed also by the compact design of the structure, which has been studied to optimise the overall dimensions of the bed.

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With the user-friendly Traction Control Switch you can control both the forward/backward movement and the walking speed.

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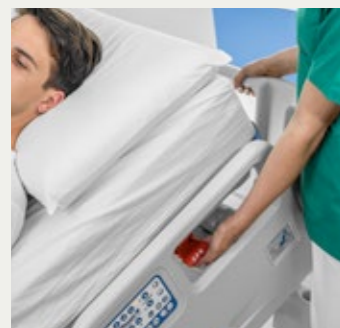
CPR
The CPR control on both sides allows the bed platform to be brought to the horizontal position immediately; accessible even with the side rail lowered.

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EN PATIENT WEIGHT STORAGE

From the touch screen it is possible to save the patient's weight data for later reference.

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EN PREDICTIVE EXIT ALERT

By detecting the patient's movements, it is possible to receive an alert when the patient is about to leave the bed unattended.

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EN RESTLESS PATIENT ALERT

The corner weighing sensors are able to detect different levels of patient movement, alerting the operator if the patient becomes restless.

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EN PATIENT EXIT ALERT

The weight monitoring system can be programmed to alert if the patient leaves the bed.

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An entire **operating platform in a bed**

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EN DELTA4 is the most popular Malvestio bed. A versatile bed that adapts to the needs of each department: the bed becomes an operating platform, customisable to provide the best assistance within any context. Perfect as an inpatient bed, it can be configured according to the specific needs of various wards. Controls are available for different solutions: integrated push-button panels on the side rails, supervisor at the foot of the bed wired push-button panel for the patient and footswitch. Delta4 can be supplied with **WEIGHING SYSTEM**. **DELTA4** can be supplied with the **SISTEMA INCENTRO**, which prevents the patient from slipping. The **DRIVESYSTEM**, with central electromotive wheel, can be integrated to facilitate the operator in moving the patient between wards. The removable bed frame and simple access to all its parts allow for easy and efficient sanitization.

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EN The bed
that simplifies
operators' work

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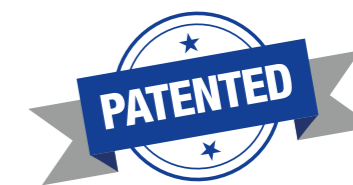
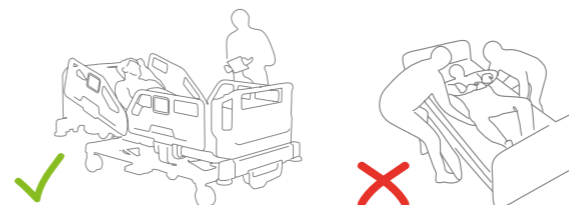
EN With the **SISTEMA INCENTRO**, the combined backrest-pelvis transfer cancels horizontal thrusts, prevents slipping and keeps the torso at an inclination of 30°. The correct positioning of the patient reduces the risk of decubitus due to "rubbing" between skin and the surface of the mattress. A non-slipping patient reduces fatigue and the risk of injury, and frees up staff to focus on activities of greatest value to patients.

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EN The exclusive weighing system was designed to guarantee a **high level of precision** independently from the mattress platform's position. Thanks to the 4 sensors located at the corners of the bed, the load is better distributed and **the reading level more accurate**. The weighing data can be recorded manually whenever necessary or automatically every 24 hours. The weighing system allows to set up different types of alerts: exceeded threshold weight alert or "bed exit" alert.

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More manageable to transport
the patient effortlessly and safely

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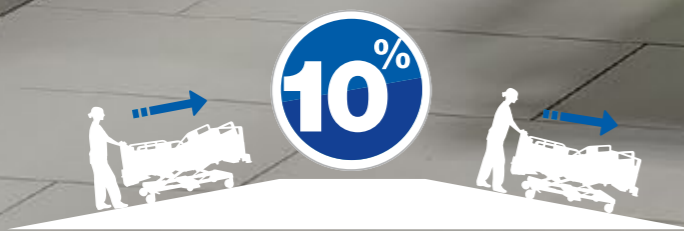
EN Thanks to the DRIVESYSTEM ascent support and descent assisted braking system, **ramps with inclines of up to 10% can be navigated easily and safely.**

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EN With the user-friendly Traction Control Switch you can control both the forward/backward movement and the walking speed.

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EN The system is equipped with an anti-crushing sensor that ensures the operator's safety during backwards maneuvers.

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Quality as the key to competitiveness in healthcare

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EN With a commitment to responsibility at the heart of our Care Manifesto, we believe that quality should be an integral part of a company's strategic decisions. A responsibility that not only means product and service quality, but also a duty to improve our company every day. This is why we were one of the first in our industry to obtain **ISO 9001 and ISO 13485 Quality Management System Certification**. As a company like ours, quality is the tool to increase competitiveness and promote healthy growth. Above all, it translates into customer satisfaction and an ever-improving service provided to healthcare users.

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EN QUALITY MANAGEMENT SYSTEMS - MEDICAL DEVICES

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EN OCCUPATIONAL HEALTH AND SAFETY MANAGEMENT SYSTEMS

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EN MANAGEMENT SYSTEMS FOR GENDER EQUALITY

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EN ENVIRONMENTAL MANAGEMENT SYSTEMS

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The products of MALVESTIO S.p.A. are subject to continuous improvements, for this reason the technical and/or aesthetic characteristics of the product may be changed without notice.

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