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IT - EN
ES - FR

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EN Welcome to the world of Malvestio, the Italian company that has been at the forefront of hospital innovation for more than 80 years with a team of 200 professionals and a presence in over 70 countries around the world. A company that is growing in size guided by a clear corporate vision: to offer healthcare increasingly advanced technologies, high production standards, organizational efficiency, high quality products and services.

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EN All Malvestio products, are created in accordance with the principles of the **Care Manifesto**, with which we have made our commitment to efficient healthcare focused on the quality of life of the patient. In particular, **"CARE FOR QUALITY"** is the duty we take on towards the efficiency of our work and our products, with which we contribute to the progress of healthcare and society.

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A **WORKING TOOL** DESIGNED FOR INTENSIVE CARE PATIENTS AND OPERATORS

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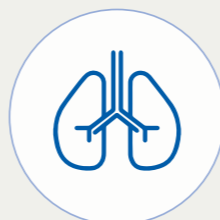
EN ERGONOMICS, SAFETY AND RISK REDUCTION: electric handling allows optimal patient positions.

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EN MANAGEMENT AND PREVENTION OF POLMONARY COMPLICATIONS: postural variations, chest tilt and planned rotational movement promote lung recovery.

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EN CONTINUOUS PATIENT MONITORING: advanced technology and bedside innovation for monitoring and communication with medical staff.

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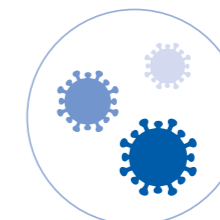
EN PREVENTION OF DECUBIT DISEASE IN CRITICAL PATIENTS: repositioning strategies and techniques to reduce the incidence of skin lesions.

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EN CONTROL AND PREVENTION OF NOSOCOMIAL DISEASES: designed to ensure the necessary sanitisation in the healthcare environment in a simple and efficient manner.

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The new approach to **intensive care challenges**

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EN **ATLANTE** is designed to support intensive care with the best of our technology, ensuring a quality therapeutic response. From its intuitive design to its feature set, **ATLANTE** is the choice of excellence for healthcare facilities offering the highest level of care. It is fully electric, with easy-to-reach touch screens and intuitive icon-based controls. It is designed as an integrated bed-mattress system, capable of sensing and distributing pressure according to bed inclinations. The **WEIGHING SYSTEM's** precision sensors weigh the patient in real time, providing the operator with essential organisational support.

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THE BED AS A PLATFORM FOR CRITICAL AREAS

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EN ATLANTE is the most advanced expression of our idea of the bed as an active partner in healthcare: a true operating platform for critical areas, with design, functions and materials that optimise the work of professionals and the quality of care.

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optimal position for the patient, less fatigue for the operator¹

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EN The patented PAS SYSTEM backrest movement system facilitates patient repositioning manoeuvres by optimally **combining the movements of the backrest and mattress.** This active adjustment prevents the patient from sliding towards the bottom of the bed, **reducing the risk of pressure injuries** that are often the cause of pressure ulcers. The movement of the backrest takes place simultaneously in three directions: **tilting, shifting and raising**, while the mattress remains integral with the mattress platform, **keeping the patient in the correct position.**

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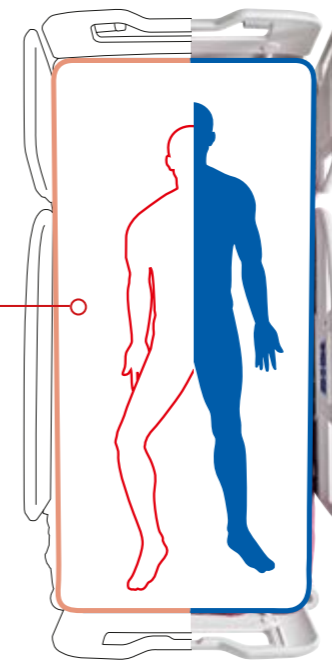
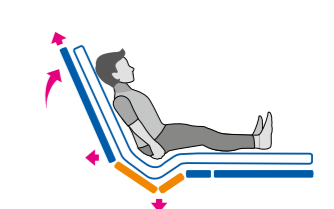
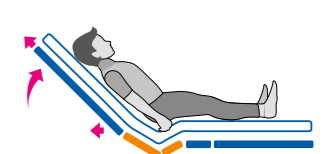
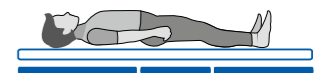
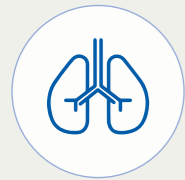
EN The inclination of the mattress platform on both sides provides dynamic support for the patient in conditions of prolonged immobility.

Customised rotation programmes can be set via the touch screen console, with configuration of the number of cycles, times and angles.

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EN Touch screen programming of **rotational system**.

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Lateral tilting: active prevention and more effective care.

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EN The **lateral tilt** of the **ATLANTE** mattress platform, which can be operated from any side of the bed allows clinical procedures, medication and other activities to be performed safely even by a single operator.

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Broad radiotransparent area
for faster and more comfortable diagnosis

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EN The broad radiotransparent area of **ATLANTE** bed allows radiological examinations to be performed without the need to move the patient.

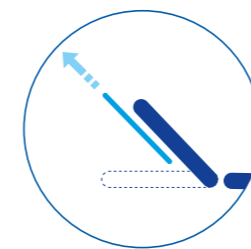
ATLANTE enables the use of the most common diagnostic imaging equipment directly from the bed. The X-ray cassette holder can also be pulled out with the backrest in a horizontal position, avoiding the need to move the patient at the end of the examination.

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EN **ATLANTE** can be equipped with an optional sliding and removable X-ray cassette holder for fast and precise positioning of the X-ray plate.

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Weight and patient movements in real time

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EN The exclusive **ATLANTE** bed weighing system guarantees high accuracy, thanks to 4 sensors arranged in the corners of the bed. This configuration ensures optimal load distribution and accurate weight reading. It is also possible to set different types of customised alerts.

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AN **EASY-MOVING BED** DESIGNED
TO ASSIST OPERATORS

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EN The compact external structure reduces lateral bulk without compromising patient comfort. This design **makes it easier to move the bed** even in difficult spaces.

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Effortless and risk-free
patient transport

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EN The **DRIVESYSTEM** ascent support and descent assisted braking system allows **to navigate easily and safely ramps with inclines of up to 10%.**

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With the Traction Control Switch you can control both the forward/backward movement and the walking speed.

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To prevent collisions while moving, **ATLANTE** is equipped with anti-collision sensors with radar vision that automatically reduce speed in the presence of obstacles.

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Tailor-made solutions
for intensive care staff

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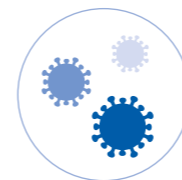
EN ATLANTE has a **touchscreen console and a physical console** with an LED-illuminated keyboard. The panels make it easy to check the controls while working from any position and in any bed position.

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EN Shapes and materials **for easy cleaning**. ATLANTE's removable mattress platform allows access to every corner of the bed, while the plastic materials are specially developed for quick and complete sanitisation.

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EN The cardiology seat configuration of **ATLANTE** promotes optimal posture for comfort during prolonged therapy.

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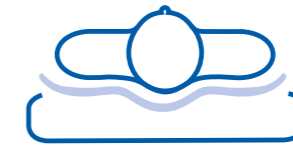
EN GLOBE* with individual pressure cells provides **active and dynamic support for the patient** by alternating contact points to distribute pressure effectively. The system works with continuous or alternating low pressure to ensure prolonged comfort and an optimal resting position.

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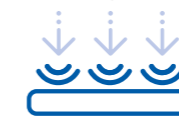
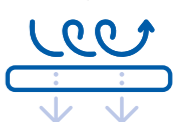
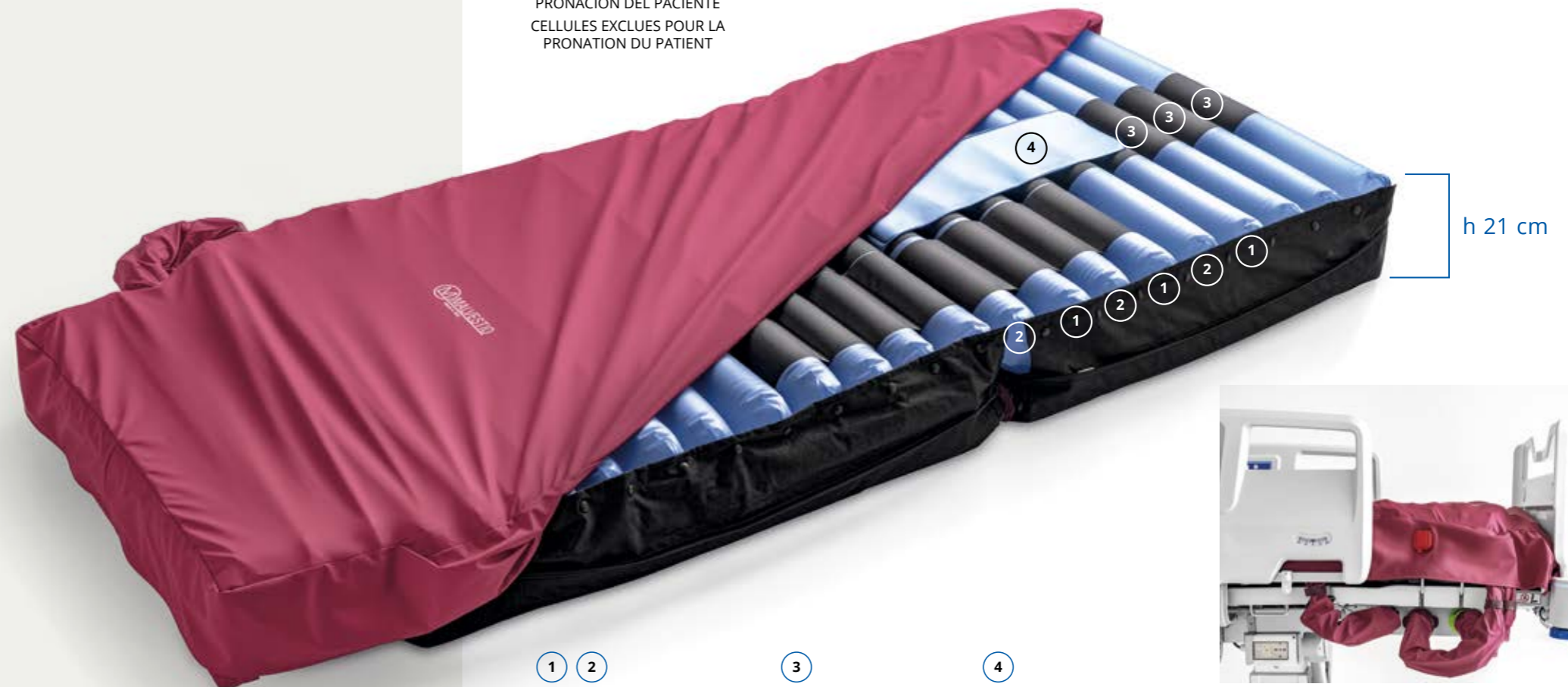
The mattress length adjusts automatically according to the bed movements.

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Intensive and **semi-intensive care**
has just become more efficient

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EN ARIA combines Malvestio technologies into a flexible bed, suitable for both intensive and semi-intensive care.

Its functions respond to the various challenges of intensive care and promote safety and comfort for patients and staff.

The large area of radiotransparency allows for radiographic testing without moving patients from bed.

With the **SISTEMA INCENTRO** it is possible to prevent the patient's body from slipping. Furthermore, through the **Lateral Tilt system**, it is easier to monitor the patient.

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

offsets the patient push towards the foot-end of the bed

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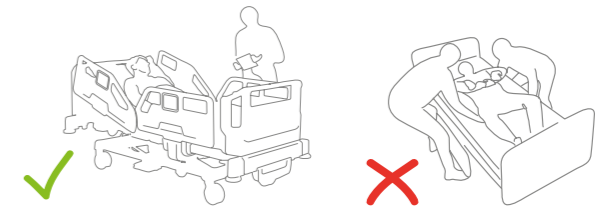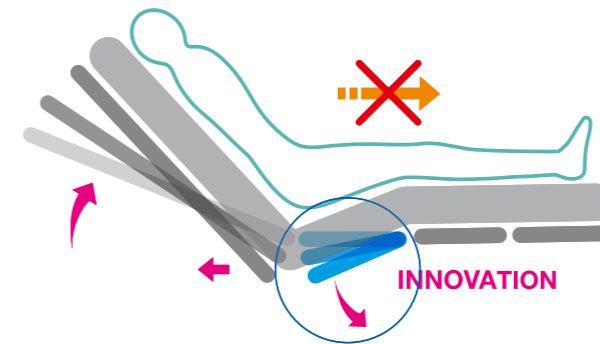**NO STRESS NO HASSLE, MORE TIME FOR
WHAT MATTERS MOST.**

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Intensive care patients need to be frequently repositioned by the nursing staff as their body tends to slip towards the foot-end of the bed, with subsequent skin friction. A non-slipping patient reduces fatigue and the risk of injury, and frees up staff to focus on activities of greatest value to patients.

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programmed rotation for patient health²

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EN The lateral tilting function can also be programmed from the touch screens to activate automatically, through the "rotational system" function. **The automatic inclination allows**, without engaging healthcare workers in strenuous activities, **for preventing complications deriving from a prolonged stay in the same position such as pressure ulcers and intrapulmonary fluid formation.**²

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**LATERAL TILT: FOR EASIER PATIENT CARE AND LESS
FATIGUE FOR PROFESSIONALS.**

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EN The lateral tilt of the mattress platform makes medical staff activities more ergonomic as the patient **can be placed on one side with the minimum effort**. This way, nursing staff can operate on the patient, preventing injuries and reducing fatigue at work. **The activation and extent of the lateral tilting can be controlled via the touch screen console, the side push button panel on the side rails and the pedal panel on both sides.**

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Controlling the patient's weight and movements in real time

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EN The exclusive weighing system was designed to guarantee a high level of precision, regardless of the position of the mattress platform. Thanks to the 4 sensors located at the corners of the bed, the load is better distributed and the reading level more accurate. The weighing system allow to set up different types of alerts.

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EN It is possible to record patient weight data and to view the corresponding graphics on the touch screen.

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EN With a large **radiotransparent area**, **ARIA** range beds are equipped for taking X-rays without having to remove the patient. The radiotransparent plastic laminate backrest features a cassette for X-ray plates that allows X-ray examinations to be carried out on a bedridden patient.

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EN **ARIA** allows for a broad area of **radiotransparency**, thus allowing for the use of the most common equipment such as the image intensifier.

The X-ray cassette holder can also be removed with the backrest in a horizontal position, thus avoiding disturbing the patient at the end of the examination.

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EN Thanks to the **DRIVESYSTEM** ascent support and descent assisted braking system, ramps with inclines of up to 10% can be navigated easily and safely.

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EN COMPACT DIMENSIONS AND EASY MANOEUVRING

The practicality and safety of manoeuvring ARIA range beds are guaranteed also by the compact design of the structure, which has been studied to optimise the overall dimensions of the bed.

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With the user-friendly Traction Control Switch you can control both the forward/backward movement and the walking speed.

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EN CPR

The CPR control on both sides allows the bed platform to be brought to the horizontal position immediately; accessible even with the side rail lowered.

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EN PATIENT WEIGHT STORAGE

From the touch screen it is possible to save the patient's weight data for later reference.

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EN PREDICTIVE EXIT ALERT

By detecting the patient's movements, it is possible to receive an alert when the patient is about to leave the bed unattended.

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EN RESTLESS PATIENT ALERT

The corner weighing sensors are able to detect different levels of patient movement, alerting the operator if the patient becomes restless.

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EN PATIENT EXIT ALERT

The weight monitoring system can be programmed to alert if the patient leaves the bed.

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An entire **operating platform in a bed**

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EN DELTA4 is the most popular Malvestio bed. A versatile bed that adapts to the needs of each department: the bed becomes an operating platform, customisable to provide the best assistance within any context.

Perfect as an inpatient bed, it can be configured according to the specific needs of various wards. Controls are available for different solutions: integrated push-button panels on the side rails, supervisor at the foot of the bed wired push-button panel for the patient and footswitch. Delta4 can be supplied with **WEIGHING SYSTEM**.

DELTA4 can be supplied with the **SISTEMA INCENTRO**, which prevents the patient from slipping. The **DRIVESYSTEM**, with central electromotive wheel, can be integrated to facilitate the operator in moving the patient between wards.

The removable bed frame and simple access to all its parts allow for easy and efficient sanitization.

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EN With the **SISTEMA INCENTRO**, the combined backrest-pelvis transfer cancels horizontal thrusts, prevents slipping and keeps the torso at an inclination of 30°.

The correct positioning of the patient reduces the risk of decubitus due to "rubbing" between skin and the surface of the mattress. A non-slipping patient reduces fatigue and the risk of injury, and frees up staff to focus on activities of greatest value to patients.

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EN The exclusive weighing system was designed to guarantee a **high level of precision** independently from the mattress platform's position. Thanks to the 4 sensors located at the corners of the bed, the load is better distributed and **the reading level more accurate**. The weighing data can be recorded manually whenever necessary or automatically every 24 hours. The weighing system allows to set up different types of alerts: exceeded threshold weight alert or "bed exit" alert.

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More manageable to transport
the patient effortlessly and safely

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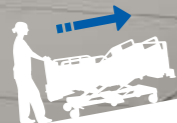
EN Thanks to the DRIVESYSTEM ascent support and descent assisted braking system, **ramps with inclines of up to 10% can be navigated easily and safely.**

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EN With the user-friendly Traction Control Switch you can control both the forward/backward movement and the walking speed.

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EN The system is equipped with an anti-crushing sensor that ensures the operator's safety during backwards maneuvers.

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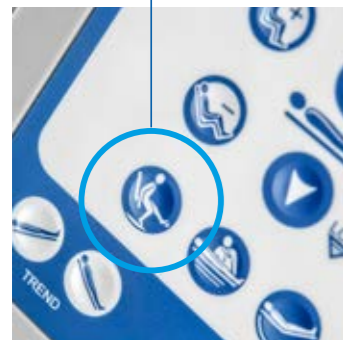
Patients are **safe** even when unattended

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EN **Safe exit position** button brings the bed to the minimum height to help the patient stand up. The patient can autonomously exit from the bed or be attended by the nursing staff.

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4 SECTOR SAFETY SIDES WITH ELECTRIC CONTROLS

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EN The system of **4 sector safety sides with integrated controls** allows full autonomy of movement to the patient.

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EN **High containment safety sides** (41 cm) with dampened descent.

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EN The X-ray tray together with HPL backrest allows extremely easy x-ray operations to be carried out

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EN The **CPR** control on both sides allows the bed platform to be brought to the horizontal position immediately; accessible even with the side rail lowered.

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REMOVABLE HEADBOARD AND FOOTBOARD

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EN The bed ends can be easily removed for emergency operations.

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EN The bed lengthener can be controlled by just one person.

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Customization and colours for every context

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EN DELTA4 bed is available in different colours and can be personalized with the Health Care facility logo.

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Configurazioni - Configurations - Configuraciones - Configurations

Mod.	EN
378300	I.C.U. electric bed
378300B	I.C.U. electric bed with weighing system
378300SB	smart I.C.U. electric bed with weighing system

ARIA

Dimensioni esterne	Overall dimensions	Dimensiones totales	Dimensions extérieures	220x99 cm
Dimensioni piano rete	Mattress platform dimensions	Dimensiones plataforma de colchón	Dimensions du plan de couchage	200x88 cm
Altezza piano rete	Mattress platform height	Altura plataforma de colchón	Hauteur plan de couchage	min 46 cm max 82 cm
Estensione allunghetto	Bed extension	Extensión para alargar la cama	Extension du plan de couchage	30 cm
Contenimento sponde da piano rete	Bed rails containment from bed frame	Contención de las barandillas desde la plataforma de colchón	Hauteur protection barrières (sans matelas)	45 cm
Inclinazione Trendelenburg e ControTrendelenburg	Trendelenburg /Reverse-Trendelenburg	Inclinación Trendelenburg /Contra-Trendelenburg	Déclive/Proclive	13° / 13°
Inclinazione schienale	Backrest inclination	Inclinación respaldo	Inclinaison section buste	0° / 63°
Inclinazione basculamento	Tilting angle	Inclinación lateralización	Latéralisation	18° / 18°
Carico lavoro sicuro	Safe working load	Carga de trabajo segura	Charge de fonctionnement en sécurité	300 kg

Configurazioni - Configurations - Configuraciones - Configurations

Mod.	EN
3900	electric bed, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system.
3900B	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system.
3905	electric bed, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system. headboard connected to the base frame.
3905B	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system. headboard connected to the base frame.
3960	electric bed, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels, electric leg rest. lateral tilting. pedal controls.
3960B	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels, electric leg rest. lateral tilting. pedal controls.
3965	smart electric bed, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels, electric leg rest. lateral tilting. pedal controls. headboard connected to the base frame.
3965B	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels, electric leg rest. lateral tilting. pedal controls. headboard connected to the base frame.
3900SB	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels. smart functionality.
3905SB	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels. smart functionality. headboard connected to the base frame.
3960SB	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels, electric leg rest. lateral tilting. pedal controls. smart functionality.
3965SB	electric bed with weighing system, translating laminate backrest, electric Trendelenburg and Reverse-Trendelenburg, integrated bed lengthener, incenter system, 4 section side rails with electric controls and touch screen control panels, electric leg rest. lateral tilting. pedal controls. smart functionality. headboard connected to the base frame.

DELTA4

Dimensioni esterne	Overall dimensions	Dimensiones totales	Dimensions extérieures	220x99 cm
Dimensioni piano rete	Mattress platform dimensions	Dimensiones plataforma de colchón	Dimensions du plan de couchage	200x88 cm
Altezza piano rete	Mattress platform height	Altura plataforma de colchón	Hauteur plan de couchage	min 38 cm max 79 cm
Estensione allunghetto	Bed extension	Extensión para alargar la cama	Extension du plan de couchage	30 cm
Contenimento sponde da piano rete	Bed rails containment from bed frame	Contención de las barandillas desde la plataforma de colchón	Hauteur protection barrières (sans matelas)	41 cm
Inclinazione Trendelenburg e ControTrendelenburg	Trendelenburg /Reverse-Trendelenburg	Inclinación Trendelenburg /Contra-Trendelenburg	Déclive/Proclive	16° / 16°
Inclinazione schienale	Backrest inclination	Inclinación respaldo	Inclinaison section buste	0° / 63°
Carico lavoro sicuro	Safe working load	Carga de trabajo segura	Charge de fonctionnement en sécurité	300 kg

Configurazioni - Configurations - Configuraciones - Configurations

Mod.	EN
3700	electric bed, polypropylene mattress platform, translating backrest, electric Trendelenburg and Reverse-Trendelenburg.
3705	electric bed, polypropylene mattress platform, translating backrest, electric Trendelenburg and Reverse-Trendelenburg, headboard connected to the base frame.
3710	electric bed, polypropylene mattress platform, translating backrest, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section siderails
3710B	electric bed, polypropylene mattress platform, translating backrest, bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section siderails. integrated weighing system.
3715	electric bed, polypropylene mattress platform, translating backrest, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section siderails. headboard connected to the base frame.
3715B	electric bed, polypropylene mattress platform, translating backrest, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section siderails. headboard connected to the base frame. with integrated weighing system
3720	electric bed, polypropylene mattress platform, translating backrest, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section side rails with integrated controls.
3720B	electric bed, polypropylene mattress platform, translating backrest, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section side rails with integrated controls. with integrated weighing system
3725	electric bed, polypropylene mattress platform, translating backrest, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section side rails with integrated controls. headboard connected to the base frame.
3725B	electric bed, polypropylene mattress platform, translating backrest, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section side rails with integrated controls. headboard connected to the base frame. with integrated weighing system
3720SB	smart electric bed compatible with ermes system, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section side rails with integrated electrical controls and touch screen control panels, weighing system
3725SB	smart electric bed compatible with ermes system, integrated bed lengthener, electric Trendelenburg and Reverse-Trendelenburg, complete with 4-section side rails with integrated electrical controls and touch screen control panels, headboard connected to the base frame, weighing system.

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The coating that actively **protects** surfaces

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EN MICROFREE® is a powder coating with **sanitizing properties** that continuously and permanently reduces the presence of microorganisms on surfaces. This product represents an **advanced technological solution in the powder coating sector**, designed to support health safety and reduce the risk of bacterial contamination on frequently touched or exposed surfaces.

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EN MICROFREE® reduces the bacterial load on surfaces by over 99% within a few hours of application, according to accredited laboratory tests.

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EN The sanitizing properties of MICROFREE® are automatically activated as soon as the paint is applied and act continuously, reducing the need for manual cleaning. This means an active and continuous hygienic benefit on the painted surface.

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EN The sanitizing function of MICROFREE® resists normal cleaning cycles and remains effective throughout the life cycle of the paint.

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EN MICROFREE® is biocompatible and suitable for direct contact with human tissue, ensuring safety even in applications that require high standards of skin compatibility.

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Quality as the key to competitiveness in healthcare

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EN With a commitment to responsibility at the heart of our Care Manifesto, we believe that quality should be an integral part of a company's strategic decisions. A responsibility that not only means product and service quality, but also a duty to improve our company every day. This is why we were one of the first in our industry to obtain **ISO 9001 and ISO 13485 Quality Management System Certification**. As a company like ours, quality is the tool to increase competitiveness and promote healthy growth. Above all, it translates into customer satisfaction and an ever-improving service provided to healthcare users.

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EN OCCUPATIONAL HEALTH AND SAFETY MANAGEMENT SYSTEMS

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EN ENVIRONMENTAL MANAGEMENT SYSTEMS

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EN QUALITY MANAGEMENT SYSTEMS - MEDICAL DEVICES

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EN MANAGEMENT SYSTEMS FOR GENDER EQUALITY

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The products of MALVESTIO S.P.A. are subject to continuous improvements, for this reason the technical and/or aesthetic characteristics of the product may be changed without notice.

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